EN	Instructions for use	2
DE	Gebrauchsanweisung	10
FR	Mode d'emploi	18
IT	Istruzioni per l'uso	26
NL	Gebruiksaanwijzing	34
SW	Bruksanvisning	42
FI	Käyttöohjeet	50
NO	Bruksanvisning	58
SP	Instrucciones de uso	66
PO	Manual de instruções	72

I Congratulations

The Vario is a high-quality secretion aspirator. The QuatroFlex™ suction aggregate is the innovative continuation of the proven Medela piston / cylinder system. User-friendliness, simple cleaning and safety features are additional advantages of these versatile secretion aspirators. A comprehensive range of accessories makes the Vario ideally suited to a wide range of medical applications. Contact us – we will be pleased to advise you!

II Use in accordance with the instructions

The device is indicated for use for the aspiration and removal of surgical fluids, tissue (including bone), gases, bodily fluids or infectious materials, either in the emergency room, in the doctor's practice and at the patients bedside for:

	Vario 8	Vario 8 AC/DC	Vario 8 c/i*	Vario 18	Vario 18 AC/DC	Vario 18 c/i*
General suction purposes	●	●	●	●	●	●
Thoracic drainage	●	●		●	●	●
Wound drainage				●	●	●
ENT indications				●	●	●
Gastric suction						●

*c = constant suction / i = intermittent suction

An extensive range of accessories makes the Vario the ideal device for a broad range of medical applications. Contact us – we will be pleased to advise you.

III Safety instructions

Correct and safe operation only with Medela accessories

Read all safety instructions before use.

- **Warning!** For use only by medically trained persons who have been adequately trained in suction procedures and in the use of aspirators. Only for medical suction purposes. Incorrect use can cause serious injury.
- Before you plug in the device, please check that your local power supply is the same as the voltage given on the specification plate.
- In each of the following cases, the device must not be used and it must be repaired by Customer Services:
 - If the power cable or the plug are damaged
 - If the device is not functioning perfectly
 - If the device is damaged
 - If the device shows clear safety defects.

- The overflow protection/bacteria filter protects the pump against overflow. If this filter is wet or blocked, the suction process is interrupted. The filter must be tested before each use and periodically during use of the device (see Chapter X).
- The pump must stand upright during use.
- The device must not be used for suctioning explosive, easily flammable or corrosive liquids
- The connecting tubing supplied with the device must never come into direct contact with the suction area. A sterile suction catheter must always be used (risk of infection).
- In the case of overflow, inform the internal technical service immediately and perform the tasks in the service manual.
- Keep the power cord away from hot surfaces.
- The plug and the ON/OFF switch must not come into contact with moisture.
- Separation from the mains is only assured through the disconnection of the plug and socket connection.
- Never pull the plug out of the mains socket by pulling on the connecting cable!
- Never use the device at high room temperatures, while bathing or showering, if you are very tired or in an environment where there is a risk of explosion.
- Never leave the device unattended when it is switched on.
- Never place the device in water or other liquids.
- The Medela Vario is a medical device that requires special safety measures in regard to EMC. It must be installed and put into operation in accordance with the attached EMC information. Portable and mobile RF communications equipment can affect medical devices.

Applications


- Thoracic drainage** When using the Vario 8 (low vacuum) for thoracic drainage, a safety device, e.g. surge chamber, is absolutely essential.
When using the Vario 18 (high vacuum) for thoracic drainage, a safety device, e.g. surge chamber in combination with a low vacuum regulator is absolutely essential.
When using an outside product in combination with the Vario the manufacturer data must be observed.
- Wound drainage** If the Vario is used for wound drainage, care must be taken to ensure that the wound has not dried out and that the bandage is firmly in place. If an unusually large amount of fluid collects in the suction jar (excessive loss of blood), the wound drainage must be stopped.
- Gastric suction** Only with Vario 18c/i: When the device is used for gastric suction, the patient's condition and correct functioning of the pump (on the basis of acoustic and visual signs) must be checked at least once every four hours.
- ENT indications** For patients for whom a breakdown of the device can lead to a critical situation (e.g. patients with acute dyspnoea or severe catarrhal congestion), a replacement device must always be available.

These Instructions for Use must be kept for later reference.

IV Guarantee

Two years guarantee for the device, and six months guarantee for the rechargeable battery and the Quatroflex™ suction unit. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage as well as use by unauthorized persons.

V Operating elements

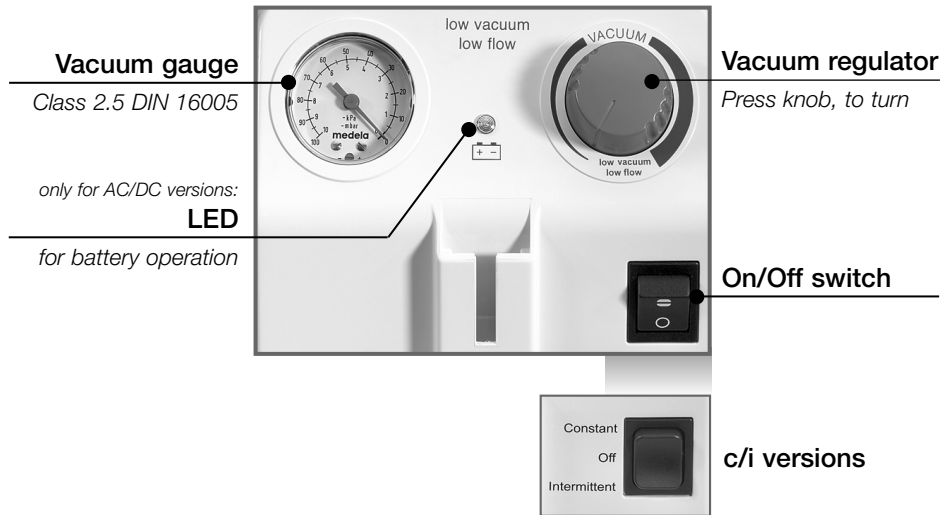Other elements (see inside of cover):

- c** Adaptor (lateral) for second suction jar
- d** Adaptor (at front) for suction jar
- e** Housing of the device
- f** Handle
- g** Tube holder
- h** 12 V DC connector socket (only AC/DC models)

On the bottom of the device:

- i** Exhaust
- j** Specification plate
- k** Fuse

VI Accessories

Correct and safe operation only with Medela accessories

Contained in the delivery package

- a** 077.0931 Silicone tubing with coupling and angle pieces
- b** 077.0571 Overflow protection / bacteria filter (5 pcs)

Optional accessories – not contained in the delivery package

Reusable system

- ☞** 077.0110 1-litre suction jar PSU
- 077.0120 2-litre suction jar PSU
- 077.0155 0.5-litre suction jar PSU
- 077.0420 Large lid, conical patient connection
- 077.0440 Small lid, conical patient connection

Disposable system (only for Vario 18)

- ☞** 077.0210 1-litre suction jar PSU
- 077.0220 2-litre suction jar PSU
- ☞** 077.0310 1-litre disposable suction bag
- ☞** 077.0311 1-litre disposable suction bag with solidifier
- 077.0320 2-litre disposable suction bag
- 077.0321 2-litre disposable suction bag with solidifier
- ☞** 077.0410 Lid for disposable suction bag

General accessories


- ☞** 077.0951 Disposable tubing with fingertip
ø 8.3 x 5.9 mm, char. 25, length 180 cm
- 077.0821 Rail holder
- 077.0823 Universal holder
- 077.0824 Trolley plate
- 077.0825 Rail holder for thoracic drainage
- 077.0826 Holder for disposable canister
- 077.0831 Carrying bag
- 077.0832 Car connection cable for 12 V DC

Other accessories available on request – We will be pleased to advise you.

VII AC/DC versions

Indicators

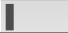	Visual 	Acoustic 	
during battery operation	does not light up		rechargeable battery is fully charged
	blinks slowly		rechargeable battery soon empty
During mains operation	lights up		rechargeable battery is being charged
	does not light up		rechargeable battery is fully charged
	blinks rapidly		rechargeable battery is defective

Performance of rechargeable battery

	low vacuum	medium vacuum	high vacuum
Duration of pumping	~120 Mins.	~ 30 Mins.	~ 30 Mins.
Charging time (charged up to 100%)	~ 5 Hrs.	~ 5 Hrs.	~ 5 Hrs.

Rechargeable battery is charged during mains operation.

Testing of rechargeable battery

	low vacuum	medium vacuum	high vacuum
1. Pull out mains plug	-90 mbar	-55 kPa	-55 kPa
2. Set required vacuum	-9 kPa	-413 mmHg	-413 mmHg
3. Plug the suction port			
4. Switch on the Vario			
5. Measure the running time			
6. Compare the running time	> 120 Mins.	> 30 Mins.	> 30 Mins.

ok?

OK, if the above values are reached

not OK

- Repeat the test
- Calibrating the rechargeable battery

NiMH rechargeable battery

Before first use

Calibrating the rechargeable battery

A Completely discharge the rechargeable battery

- Disconnect the mains cable
- Switch on the Vario and allow to run until it comes to a standstill

B Charge the Vario

- Plug in the mains cable – the rechargeable battery is being charged
- Wait until the charging indicator goes out

Repeat this process once

The Vario is then ready for use

Operation with rechargeable battery

 Only charge the rechargeable battery when the acoustic signal sounds and the charging indicator blinks slowly

Storage


In order to keep the self-discharge at a minimum, store the Vario and the replacement rechargeable batteries at temperatures below 25°C (77°F). Repeat Steps A and B every 60-90 days

VIII Application

1. Check general condition of the device

- Is the overflow protection / bacteria filter present and OK? (Test – see Chapter X)
 - *Change the filter...*
 - once a week
 - after an overflow
 - when there is a change of patient
- Check seal of tubing connections
- Suction jars and lids – Check for cracks, brittle areas, discolouration? – exchange, if necessary
- Mechanical overflow protection must be open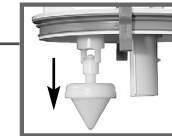

2. Test function

a. Switch the device on

- AC/DC and c/i versions: before first use, charge the rechargeable battery for five hours
- AC versions: plug in
- With AC/DC and c/i versions, select continuous pumping mode

b. Set maximum vacuum

Press knob to turn

c. Attach patient tubing and clamp tubing


d. Compare vacuum

	low vacuum	medium vacuum	high vacuum
+ 2000 m	- 7,0 kPa	- 43 kPa	- 59 kPa
+1000 m	- 7,9 kPa	- 48 kPa	- 66 kPa
+ 500 m	- 8,5 kPa	- 51 kPa	- 70 kPa
0 m	- 9,0 kPa	- 55 kPa	- 75 kPa

ok?

not OK, see Chapter XI

OK

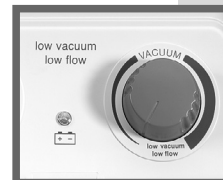3. Use

Select correct vacuum and suction mode according to the particular application)

Press knob to turn

IX Cleaning

Disconnect the device before cleaning

- Comply with in-house directives on hygiene
- Clean the device with a damp cloth and with non-aggressive cleaning material
- Suction jars, lids, suction bags and other accessories are to be handled in accordance with the instruction-sheets provided

X Test filter

Filter open!

1. Select maximum vacuum
2. Switch on the Vario
3. Read / compare the vacuum

Vario 8 low vacuum	Vario 18 medium vacuum	Vario 18 high vacuum
- 3 kPa	- 10 kPa	- 10 kPa


If the mentioned values are exceeded, the filter must be replaced.

XI Troubleshooting

- Insufficient vacuum** Check
- Setting of the vacuum regulator
 - Condition of the tubings
 - Seal of the tubings connections
 - That the overflow protection is functioning correctly
 - If the filter is blocked
 - Examine the suction jars, lids and seals for blockages / brittle areas

- Motor not running** Check
- Is mains/battery power present?
 - Is the device switched on?
 - Is the fuse ok?
- Replace fuses according to the specification plate

Before changing the fuses, it is essential to disconnect the pump from the mains supply!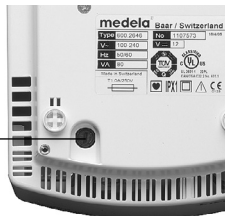

- c/i versions** Check the interval times
- Check the jumper setting
 - Correct position of the On/Off switch

On	Off
8 Secs.	4 Secs.
16 Secs.	8 Secs.
32 Secs.	16 Secs.

If the fault cannot be corrected, contact the internal technical service

XII Service work / Settings

Corresponding illustrations are to be found on the back flap.

- Disconnect the pump from the mains supply
- Remove four screws
- Remove cover

- A Replacing the rechargeable battery** 077.0833
- A1 Detach the battery cable
 - A2 Remove the rechargeable battery
 - A3 Insert new rechargeable battery
 - A4 Attach battery cable

- B Replacing QuatroFlex™** 600.0491
- B1 Remove the three tubings
 - B2 Turn QuatroFlex™ 45° to the left
 - B3 Remove QuatroFlex™
 - B4 Clean inside of the pump
 - B5 Insert QuatroFlex™
 - B6 Turn QuatroFlex™ 45° to the right
 - B7 Re-attach all the tubings

- C Set interval times**
- C1 Determine the slot required
 - C2 Attach jumper in required position

On	Off
8 sec.	4 sec.
16 sec.	8 sec.
32 sec.	16 sec.

- Attach cover
- Insert four screws
- Plug the pump in
- Only for AC/DC versions: charge the rechargeable battery for five hours
- Test functions

XIII Routine check

Routine checks and service work may only be carried out by Medela authorized personnel. The routine check is to be carried out once a year (see Service Manual)

XIV Disposal

The Vario is made from various metals and plastics. Before disposal, the device is to be rendered unusable and it must be disposed of in accordance with the local regulations. With the AC/DC versions, the electronic components and the rechargeable batteries must be disposed of separately, in accordance with the local regulations.

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